AtriClip® Gillinov-Cosgrove Selection Guide™

INSTRUCTIONS FOR USE

(CGG100)

DESCRIPTION

The AtriClip Gillinov-Cosgrove Selection Guide (Guide) is an accessory that works in conjunction with the AtriClip LAA Exclusion System with Pre-Loaded Gillinov Cosgrove Clip (hereafter: AtriClip LAA Exclusion System). The AtriClip LAA Exclusion System is comprised of the AtriClip LAA Exclusion Device (Clip), the AtriClip Applier, and the Guide.

The Guide is a single patient use surgical instrument designed to assist in the selection of the appropriate Clip. The Guide is malleable and may be placed directly adjacent to the Left Atrial Appendage (LAA) for sizing. There are markings on the guide which facilitate the evaluation of the structure and selection of the appropriate Clip size. The markings on the guide are 4mm +/- 0.5mm shorter than the clip nominal dimensions to approximate the appendage compression when the clip is applied. Tissue thickness, appendage geometry and other factors can affect sizing decisions therefore it is the physician's judgment to select the appropriate size.

The Guide does not contain natural rubber latex components.

PRODUCT COMPOSITION

The AtriClip LAA Exclusion System consists of a single use, sterile, self-closing, implantable Clip preloaded on a single use Clip Applier along with a selection guide.


PART NAME

Selection Guide

TYPE AND SPECIFICATION

CGG100

ATRICLIP GILLINOV-COSGROVE SELECTION GUIDE ILLUSTRATION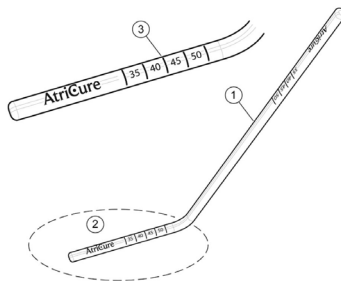

- [1] Handle
- [2] Base
- [3] Indication Marks

⚠ BEFORE USING PRODUCT READ THE FOLLOWING INFORMATION THOROUGHLY

IMPORTANT!

This booklet is designed to assist in using this product. It is not a reference to surgical techniques.

INDICATIONS FOR USE

The AtriClip Gillinov-Cosgrove™ Selection Guide (Guide) is intended to be used as an accessory to the AtriClip LAA Exclusion System.

CONTRAINDICATIONS

None known

⚠ WARNINGS ⚠

1. Do not use the Guide if the sterility barrier is breached. To avoid the risk of patient infection, inspect the product packaging prior to opening to ensure that the sterility barrier is not breached.
2. Do not apply excessive force when using the Guide. Using excessive force may cause tissue damage.


⚠ PRECAUTIONS

1. Carefully read all instructions for the AtriClip Gillinov-Cosgrove Selection Guide prior to using the device. Failure to properly follow instructions may lead to injury and/or may result in improper use of the device.
2. The AtriClip Gillinov-Cosgrove Selection Guide is to be used only to assist with selecting the appropriate Clip.
3. Use of the Guide should be limited to properly trained and qualified medical personnel.
4. To avoid damage to the device, do not drop or toss the Guide. If the Guide is dropped, do not use. Replace with a new Guide.
5. To avoid damage to the Guide, use caution when handling and interfacing with other devices.
6. Do not bend the guide in the area of the indication marks.
7. Do not re-sterilize or reuse the Guide. Single Patient use only.

INSTRUCTIONS FOR USE

1. Using sterile technique, remove the Guide from its packaging.
2. Align the Guide adjacent with the LAA.

NOTE: Evaluation should occur at the base of the appendage.

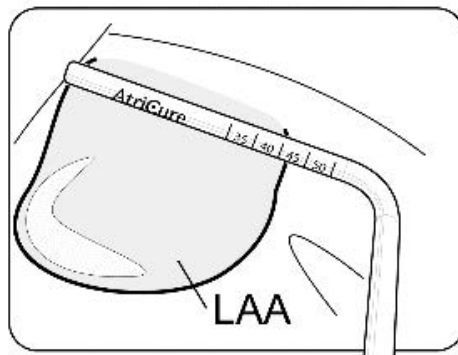Clip Size	LAA Size Range
35 mm	29 - 35 mm
40 mm	34 - 40 mm
45 mm	39 - 45 mm
50 mm	44 - 50 mm

3. Evaluate the LAA and select the appropriate Clip size.
4. Discard the Guide after use. Follow the local governing ordinances and recycling plans regarding disposal or recycling of device components.

HOW SUPPLIED

The AtriClip Gillinov-Cosgrove Selection Guide is supplied as a STERILE instrument and is for single patient use only. Sterility is guaranteed unless the package is opened or damaged. Do not re-sterilize.

Do not use the Gillinov-Cosgrove Selection Guide after the expiration date indicated.

RETURN OF USED PRODUCT

If for any reason this product must be returned to AtriCure, Inc., a return goods authorization (RGA) number is required from AtriCure, Inc., prior to shipping.

If the product has been in contact with blood or body fluids, it must be thoroughly cleaned and disinfected before packing. It should be shipped in either the original carton or an equivalent carton, to prevent damage during shipment; and it should be properly labeled with an RGA number and an indication of the biohazardous nature of the contents of shipment.

Instructions for cleaning and materials, including appropriate shipping containers, proper labeling, and an RGA number may be obtained from AtriCure, Inc.

⚠ CAUTION: It is the responsibility of the health care institution to adequately prepare and identify the product for shipment.

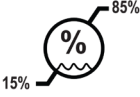
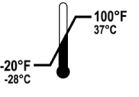
DISCLAIMER STATEMENTS

Users assume responsibility for approving the acceptable condition of this product before it is used, and for ensuring that the product is only used in the manner described in these instructions for use, including, but not limited to, ensuring that the product is not re-used.

Under no circumstances will AtriCure, Inc. be responsible for any incidental, special or consequential loss, damage, or expense, which is the result of the deliberate misuse or re-use of this product, including any loss, damage, or expense which is related to personal injury or damage to property.

EXPLANATION OF SYMBOLS ON PACKAGE LABELING

Refer to the outer package label to see which symbols apply to this product.

	Non-pyrogenic		Use-by date
	Sterilized by Irradiation		Caution
	Single Use Only		Consult Instructions For Use
Rx ONLY	Caution: U.S. Federal law restricts this device to sale by or on the order of a physician or other licensed practitioner		Manufacturer
	Lot Number		Do not use if package is damaged
	Do not re-sterilize		Does Not Contain Natural Rubber Latex
 Humidity Limitation		 Temperature Limitation	

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1-866-349-2342, 1-513-755-4100

LEGAL AGENT

AtriCure (Beijing) Medicine Information Consulting Service co Ltd.

ADDRESS

Room 418, Floor 4, Unit 2, Building 1, No. 3 Courtyard, Jinhang Middle Road,
Shunyi District, Beijing (Tianzhu Comprehensive Bonded Zone)

TELEPHONE

010-84554569

NAME OF AFTER-SALES SERVICE AGENT

AtriCure (Beijing) Medicine Information Consulting Service co Ltd.

ADDRESS OF AFTER-SALES SERVICE AGENT:

Room 418, Floor 4, Unit 2, Building 1, No. 3 Courtyard, Jinhang Middle Road, Shunyi District, Beijing (Tianzhu Comprehensive Bonded Zone)

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